

4). Check that the thermoplastic hand plate is covering all large knuckle joints (MPs) of the fingers. The inside strap should go around all the fingers to the palm side and attaches to the thermoplastic base near the small finger. This is important for maintaining close contact of the thermoplastic hand plate against the back of the hand. This strap can be adjusted by tightening or loosening the Velcro® tab near the small finger.



5). Fasten the wrist strap at a slight angle so that close contact is maintained with the thermoplastic hand plate against the back of the hand.



Additional covers can be ordered online at www.Ready-Steady.com



READI STEADI®
ANTI-TREMOR ORTHOTIC GLOVE SYSTEM